


SPLAYOMETER

Quick Start Guide

Thank you for your purchase of the Splayometer. Please read this guide before using the device. The full user manual is available online at <https://www.splayometer.com>


Attachments

The hand loop and finger rings connect to the load cell via a 90 degree twist lock. Push them together and then turn:

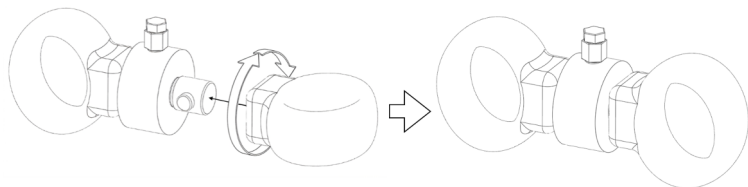

Getting Started

Flip the power switch On. The “Splayometer” screen appears and shows the battery level, then the device shows the default measurement mode.

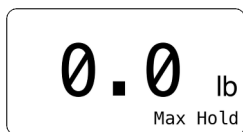

NOTE: The Splayometer goes to sleep after 15 minutes. Press any button to wake.

Buttons

Button	Short Press	Hold (2 seconds)
ZERO	Zero the scale	Open the Settings screen
MODE	Change the mode (Live → Max Hold → Average)	Switch the units (kg ↔ lb)
CAL	Average mode only: Advance step	Enter Calibration mode

Taking a Measurement

1. Press **ZERO** to zero the scale with no load applied
2. Apply force (splay or pinch) to the load cell
3. Read the value on the screen

Measurement Modes

Live — Reads applied force continuously. Press ZERO to zero the scale.


Max Hold — Captures the peak applied force. Press ZERO to reset.

NOTE: The Splayometer was factory set to Max Hold mode; can be changed in Settings.

Average — Records three-trials and averages results:

1. Press **CAL** to begin
2. Apply force, press **CAL** to save Trial #1
3. Repeat for Trials #2 and #3
4. After Trial #3, the mean result is displayed
5. Press **CAL** again to reset and start over

Settings Screen Overview

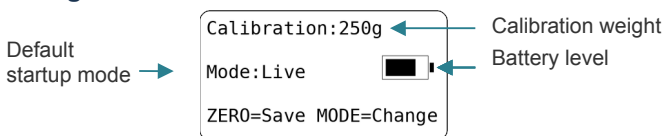

NOTE: The Splayometer was factory-calibrated and tested with 500g.

Full User Manual

See the full manual for details: <https://splayometer.com/pages/manual.pdf>

The Splayometer is a medical device intended for use by licensed healthcare professionals.
This guide is for informational purposes only.

© 2026 Splayometer. All rights reserved. Patent pending.